

A black and white photograph of an Erbe VIO S electrocautery unit in an operating room. The unit is a rectangular device with a control panel featuring a digital display and several buttons. It is mounted on a stand. In the background, a surgical microscope is visible, and the overall setting is a clinical environment.

**erbe**  
power your performance.



**VIO® S**

Electrosurgical system for use in clinics  
or specialist surgical ORs

# VIO® 300 S and VIO® 200 S:

Electrosurgery tailored to perfection.

*With the VIO® electrosurgical system, Erbe has set innovative standards aimed at providing optimum surgical support for almost any discipline as well as including a range of additional indications.*

Erbe VIO® 300 S and 200 S generator modules offer automatic power adjustment for all control technologies:

- ✔ Voltage control for gentle, reproducible cutting and coagulation
- ✔ Arc control for high-energy cutting or coagulation and for cutting under water
- ✔ Power control to maintain constant power levels during coagulation and devitalization

---

## TAILORED PRECISELY TO YOUR NEEDS – BOTH IN TERMS OF HARDWARE AND SOFTWARE

---

- ✔ Automatic power adjustment
- ✔ New and enhanced CUT and COAG functions, monopolar and bipolar
- ✔ Can be configured for custom setups based on the specific discipline, indication or procedure
- ✔ Simple, interactive and safe operation
- ✔ The VIO® S models – the master control units for other modules in the VIO® electrosurgical system, for example argon plasma coagulation, smoke plume evacuation, the endoscopy irrigation pump, and other components



# Versatile operating and safety concept that offers complete convenience.

## Simple operating concept

The operational design of the VIO® 300 S enables fast, direct access to the program parameters. Using the up / down buttons, power values and effects can be easily and directly adjusted.

## Consistent enhancement: the NESSY® patient plate safety system

With the NESSY® safety concept and the Erbe patient plate NESSY® Ω, VIO® sets new standards with regard to the safety of monopolar electrosurgery.

## Preselectable effect settings

For consistent surgical results with reproducible tissue effects.



For use in clinic ORs or specialist surgical ORs


	VIO® 300 S	VIO® 200 S
Gynecology	■	-
Urology	■	-
General surgery	■	-
Gastroenterology / Endoscopy	■	■
Pulmonology	■	■
ENT	■	-
Orthopedics	■	-
Dermatology	-	■
OMS	■	-
Ophthalmology	-	■

### Legend:

- strongly recommended
- recommended

# CUT

Precise cutting using these modes.

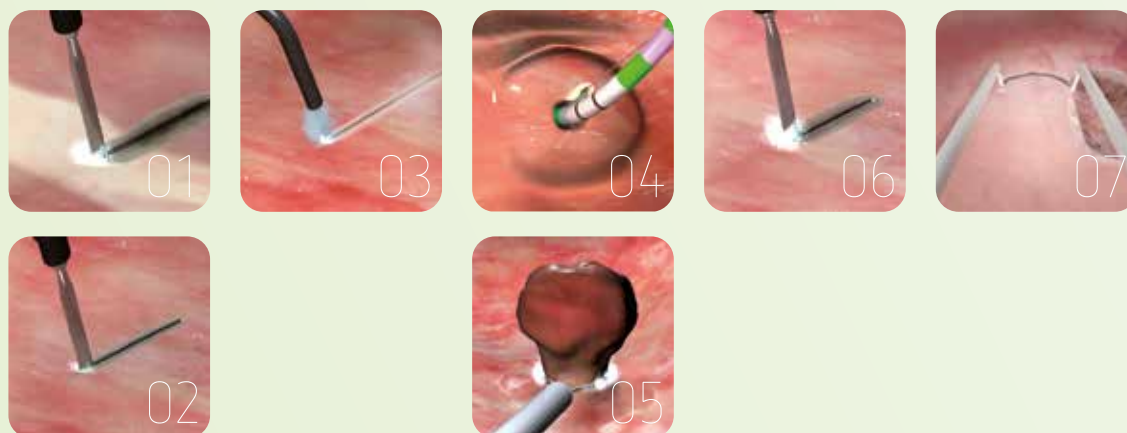

## HIGH CUT 01

Suitable for cutting inside fatty structures or under water (e.g. TUR). Strong hemostasis at the incision edges. Control of arc intensity.

## AUTO CUT 02

Standard mode for cutting with minimum necrosis and reproducible cutting quality.

## ARGON AUTO CUT 03

Mode for argon-supported cutting. Minimum carbonization, minimum smoke plume development. Results in a good post-operative healing process.

## ENDO CUT I 04

Fractionated cutting mode for papillotomy or other needle / wire applications in endoscopy.

## ENDO CUT Q 05

For endoscopic polypectomy with a snare. Fractionated cutting and coagulation cycles.

## DRY CUT 06

Cutting mode with pronounced hemostasis as a result of voltage control and modulated forms of current.

## BIPOLAR CUT 07

Bipolar cutting with all the advantages provided by voltage regulation in 8 predetermined cutting qualities. The cutting current is only present at the distal end of the applicator. This ensures more safety and guarantees precise cuts.

# COAG

Modes for exact coagulation and devitalization.



## CLASSIC COAG 01

Exposure mode for visceral and cardiac surgery. Exact, layer-specific exposure and dissection. Minimum carbonization of the incision edges.

## SWIFT COAG 02

Effective and fast coagulation with pronounced hemostasis that is also suitable for exposure.

## TWIN COAG 03

For simultaneous activation of two instruments with only one electrosurgical unit – consistent power output.

## FORCED APC 04

Covers the entire spectrum of all types of non-contact APC coagulation. For hemostasis during endoscopy or open surgery or for surface coagulation and devitalization.

## SPRAY COAG 05

Non-contact and efficient surface coagulation with low thermal penetration. Suitable for tissue devitalization or for stopping diffuse bleeding. Extensive carbonization effects.

## FORCED COAG 06

Fast and effective standard coagulation with moderate thermal penetration. Slight carbonization effects.

## SOFT COAG 07

Gentle coagulation with deep penetration, without carbonization, resulting in minimum adhesion of the electrode. Supported by the power control.

## BIPOLAR SOFT COAG 08

Bipolar coagulation. The low voltages used in this mode prevent the instrument from sticking and distinctly reduce tissue carbonization. Effective – with a subtle adjustment of 8 different effects.

# These modes and upgrades are available with the VIO® 300 S and VIO® 200 S models

## CUT Modes

	VIO® 300 S	VIO® 200 S
AUTO CUT	■	■
HIGH CUT	○	–
DRY CUT	○	–
DRY CUT°	○	–
BIPOLAR CUT	○	–
ENDO CUT Q	□	□
ENDO CUT I	□	□
ARGON AUTO CUT	■	■
ARGON HIGH CUT	○	–
ARGON DRY CUT	○	–
ARGON DRY CUT°	○	–

## COAG Modes

	VIO® 300 S	VIO® 200 S
SOFT COAG	○	■
SWIFT COAG	○	–
SWIFT COAG°	○	–
CLASSIC COAG	■	–
FORCED COAG	○	■
SPRAY COAG	■	–
BIPOLAR SOFT COAG	■	■
TWIN COAG	□	–
FORCED APC	■	■
ARGON SOFT COAG	○	■
ARGON SWIFT COAG	○	–
ARGON SWIFT COAG°	○	–
ARGON FORCED COAG	○	■
ARGON TWIN COAG	□	–

### Legend:

- included
- depending on the version
- only available ex works (must be ordered)

### FOR PERFECT CUTTING

- ✓ Newly-developed electrosurgical monopolar and bipolar CUT modes
- ✓ Power adjustment as a result of Erbe voltage control, for reproducible cutting
- ✓ Power adjustment as a result of Erbe arc control, for reproducible, efficient cutting in high-impedance tissue
- ✓ Additional area of application from microsurgery through to power-intensive vaporization
- ✓ Cutting results largely independent of cutting speed, shape of the electrode and tissue
- ✓ Bipolar cutting for more safety
- ✓ Power Peak System for optimum cutting behavior

### FOR PERFECT COAGULATION

- ✓ Newly-developed electrosurgical COAG effects
- ✓ Power adjustment as a result of voltage control, for reproducible coagulation with optimally adjusted power output
- ✓ Power control for fast „non-stick“ coagulation without carbonization
- ✓ Monopolar and bipolar coagulation for all your requirements
- ✓ AUTO-START and AUTO-STOP functions
- ✓ TWIN COAG: simultaneous activation of two electrodes / instruments for exposure

# Technical data

## VIO® 300 S and VIO® 200 S

### Power output

Maximum cut power (VIO® 300 S)	300 watts at 500 Ohm (with PPS, briefly 400 watts)
Maximum cut power (VIO® 200 S)	200 watts at 500 Ohm
Maximum COAG power	up to 200 watts
NE safety system	NESSY®
Frequency	350 kHz

### Power connection

Line voltage	100 V – 120 V / 220 V – 240 V ± 10 %
Power frequency	50 / 60 Hz
Line current	max. 8 A / 4 A
Power consumption in standby mode	40 watts
Power consumption at max. HF power	500 watts / 920 VA
Potential equalization connection	Yes
Power fuse	T 8 A / T 4 A
Dimensions	Width x Height x Depth 410 x 165 x 380 mm
Weight	8.8 kg



2Pin-22-28-8/4 mm,  
bipolar  
No. 20140-613



**VIO® 300 S**    **VIO® 200 S**  
No. 10140-300    No. 10140-400



3Pin-Bovie,  
monopolar  
No. 20140-622



3Pin-9/5 mm,  
monopolar  
No. 20140-623



6 mm-2Pin  
patient plate  
No. 20140-642



SEE MORE INFORMATION ON  
PRODUCTFINDER.ERBE-MED.COM

#### Important information

The information, recommendations and other details contained in this medium ("information") reflect our state of knowledge and the state of the art in science and technology at the time the medium was created. The information is of a general nature, non-binding and intended solely for general information purposes; it does not constitute instructions for use.

The information and recommendations contained in this medium do not create any legal obligations on the part of Erbe Elektromedizin GmbH and its affiliated companies ("Erbe") or any other claims against Erbe. The information does not constitute any guarantees or other information on the nature of the product; such information requires an express contractual agreement with Erbe in each individual case.

Erbe is not liable for any kind of damage resulting from following the information on this medium, regardless of any legal grounds for liability. Every user of an Erbe product is responsible for testing the respective Erbe product in advance with regard to its properties and its suitability for the intended type of use or purpose. Suitable type of use of the respective Erbe product can be found in the instructions for use for the respective Erbe product. The user is obliged to ascertain whether the instructions for use available to him/her correspond to the current status of the specific Erbe product. Devices may only be used in accordance with the instructions for use.

Information on settings, application sites, application duration and use of the respective Erbe product is based on the clinical experience of individual centers, physicians and clinical literature independent of Erbe as well as post market surveillance and empirical evidence. They constitute reference values, or Experienced Starting Settings, which must be checked by the user to ensure that they are suitable for the specifically planned application. Depending on the circumstances of a specific application, it may be necessary to deviate from the information provided. The user is responsible for checking this when using Erbe products. We wish to note that science and technology are subject to constant development as a result of research and clinical experience. Similarly, medicine and clinical application are constantly evolving as a result of research and technological development. This may also make it necessary for the user to deviate from the information provided on this medium.

This medium contains information about Erbe products that may not be approved/cleared in certain countries. The user of the respective Erbe product is obliged to inform himself/herself as to whether the Erbe product he/she is using is legally approved/cleared in his/her country and/or to what extent, if any, there are legal requirements for or restrictions on its use.

This medium is not intended for use in the USA.

Erbe Elektromedizin GmbH  
Waldhoernlestrasse 17  
72072 Tuebingen  
Germany

Phone +49 7071 755-0  
info@erbe-med.com  
erbe-med.com  
medical-videos.com